



User Manual

Senti, Software Revision 1.3

Manual Senti

Produced by PATH medical GmbH, Germering, Germany.

Printed on paper, which was produced without chlorine bleaching.

Revision 7, Valid since Firmware Version 1.3, April 2011.

Copyright

© 2011 PATH medical GmbH

Reprint – even partially – only allowed with written notice from PATH medical GmbH.

Article number: 100240

Contact: info@pathme.de, +49 89 8007 6502

All mentioned items, products, brands and trademarks are registered/owned by the mentioned companies.

This manual and mentioned technical details are subject to change.

Errors and omissions excepted.

Contact information from your distributor, contact information from your service partner:

Table of contents

1 Scope of application	5
2 Remarks, used symbols	7
2.1 Notes on safety	7
2.2 Notes on operational concept	8
2.3 About this manual and further sources of information	8
2.4 Symbols and structure of the graphical user interface	9
3 Start, reset, charging, and connecting sockets	19
3.1 On / off switch - special function for showing footer	19
3.2 Hardware reset – device is stalled	20
3.3 Connecting to the sockets of the device	20
Blue socket	20
Grey socket	21
Serial interface cable	21
3.4 Charging the device and and connecting to the label printer	22
Power supply	22
Connect to label printer	22
4 MIRA – PC software and updates	23
4.1 Range of functions of the MIRA PC software	23
4.2 How to get MIRA	23
4.3 How to get updates	23
5 First steps	25
5.1 User / patient selection	25
5.2 Device settings	26
5.3 Hardware test and possible error messages	26
5.4 System information and demo mode	27
5.5 License management and upgrades	27
5.6 Other errors and their possible reasons	28
5.7 Possible error messages during the measurement of... ..	28
5.8 Cleaning Senti	28
5.9 Cleaning of headphone, accessory	28
6 Warranty, repair and service	29
6.1 Warranty	29
6.2 Repair	29
6.3 Service, routine maintenance	30
7 Accessories	31
8 Technical specifications, standards, manufacturer’s data	33
8.1 Device classification and applied standards	33

8.2 Device, storage, transport	34
Device	34
Power supply / rechargeable battery	34
Storage and transport	34
8.3 Modules	35
MAGIC	35
PTA4	35
PTA4 Extended	35
PTA3	35
SUN	35
8.4 Accessories	35
8.5 Electromagnetic compatibility report (EMC report)	36
Made in Germany by PATH medical GmbH	40

1 Scope of application

Senti is especially designed for audiological examination of children from 3 years on, for preschool- and schoolscreening, for pedaudiological diagnosis and for screening of speech understanding for school children and adults.

Before starting the measurement make sure to eliminate disturbing noise or distraction in the test room. Background noise can affect the concentration of the patient and thus compromises the test results.

A separate room with no (or not much) background noise is ideal for tests with **Senti**.

2 Remarks, used symbols

2.1 Notes on safety

This manual includes notes on safety, which need to be followed in order to allow the correct usage of **Senti**.



The connector sockets are intended for connecting to the proper plugs of the original accessories as described in section 3.3. Other devices must not be connected. During measurements with **Senti**, the serial transfer cable or the label printer cable must not be connected.

Strong electromagnetic radiation may affect the operability of the instrument. Do not use **Senti** nearby devices with strong electromagnetic radiation. Please refer to the suggestions in section 8.5.

Cleaning instructions are described in chapter 5.8 and 5.9. Accessories' cleaning instructions are described on the respective data sheets.



Following art. 1, §18 and Art. 2 of the law concerning the rearrangement of waste legislation product stewardship for batteries and rechargeable batteries from June 25th 2009:

The device includes a NiMH rechargeable battery pack.

In case the rechargeable battery pack cannot be charged anymore, the rechargeable battery pack must be replaced by an authorized distributor. The distributor is responsible for the correct disposal and storage. In case of disposal of the device, the device is not intended for consumer waste but for special waste.

A fully charged and completely functional battery pack will allow for measurements of up to 6 to 8 hours (dependent on usage).

2.2 Notes on operational concept



After turning on the device, **Senti** can be operated via a touch-sensitive display (touch screen) providing several menus and functions. Context-sensitive help screens, which explain the currently available symbols and their functions, allow an intuitive handling of the hand-held device. The context-sensitive help screens are available via the blue information icons, which are displayed on each screen in the footer at the right-hand side.



At some screens, there is an additional information icon, which will provide further information for the user.

2.3 About this manual and further sources of information

In this manual you will find information about the handling of the device as well as information about the operation and cleaning. Further information and details about the measurement modules, potential clinical applications and recommendations for combining several test procedures are explained in the guide for practical application (How-To Manual). You can download this manual from <http://www.pathme.de/support/>.





2.4 Symbols and structure of the graphical user interface





All screens contain three basic elements: the header, the main screen, and the footer.
















Fig. 1: Screen layout









The following table will provide an overview of all symbols and their corresponding function. The symbols are sorted by their appearance in one of the above elements: header, main screen and footer. The functions are also explained in the context-sensitive help on the device.

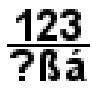


Symbols header	Meaning
<p>header structure</p>	<p>Current time menu / patient name battery level indicator</p> <p>e.g. 11:44 settings charging symbol</p>
	<p>Battery level indicator: green - sufficient power available</p>
	<p>red – charging needed.</p>
	<p>Battery is charged.</p>
	<p>Patient search pattern is active; Search pattern (filter) can be changed/deleted via magnifying glass symbol (footer).</p>



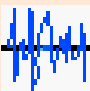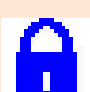

Symbols header	Meaning
Special symbols for MAGIC	Stimulus information is listed (coded) within the header. This can be deselected (hidden stimulus information) in the settings menu. Active stimulus conditions are displayed on the left, whereas information about the last recording (patient input) is displayed on the right. For further information please refer to the How-To Manual.
F / S	Frequency modulated tone / sine tone.
I / M	Instruction phase / Measurement phase.
R / L / b-R / b-L	Current stimulation at ___ ear R: right ear L: left ear b-R: Right ear (measurement of both ears selected) b-L: Left ear (measurement of both ears selected).
40 dB mute	Indication of current stimulus level (40 dB HL). Indication of a test with a muted stimulus.
e.g. 40 	Information about previous patient response: last stimulus level in dB HL (e.g. 40), patient input: tone was heard.
e.g. 60 	Information about previous patient response: last stimulus level in dB HL (e.g. 60), patient input: tone was not heard.
mute 	Patient indicated to hear a sound after presentation of a muted stimulus. Low attentiveness could be the reason for this behavior. For these responses, the respective test frequency is marked in the audiogram with a question mark (MAGIC audiogram mode only) together with the total of such events (mute - "heard").
mute 	"Not heard" patient response after presentation of a muted stimulus.




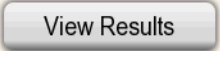



Symbols footer	Meaning
Footer structure	Back / home / turn off diverse symbols info
	Turn off the device.
	Context-sensitive help, info.














Symbols footer	Meaning
	Specific information available – slideshow on selected topics.
	Parameter settings (global or test-specific). Previously entered settings are stored for further measurements under same test conditions → individual protocols possible.
	Search patient (last name).
	Add a new patient.
	Measurement with anonymous patient (please note that data is not stored after measurement).
	Back to previous menu; Cancel data entry.
	Scroll through pages (e.g. patient list, help screen).
	Confirm data entry.
	Enter space character.
	Backspace. Delete character. The content of the edit-window can be selected (red).
	Changing between numerics, letters or special characters.






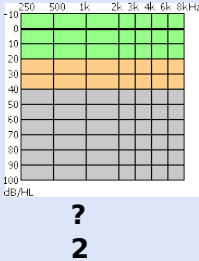



Symbols footer	Meaning
	Date input: increase number decrease number.
	Back to patient list.
	Back to main menu (i.e. test selection).
	Print test results from the test view menu (PRINTER module needed).
MAGIC Test	Special symbols during MAGIC test:
	Refill animal rack.
	Undo previous patient response.
PTA Test	Special symbols during PTA test:
	<p>Configure level shift control: shift sine level only, shift masking noise level only or simultaneously shift sine and masking noise level (locked mode).</p> <p>Shift sine level.</p> <p>Shift masking noise level.</p> <p>Simultaneously shift level of sine tone and masking noise (locked mode).</p>
	
	
	


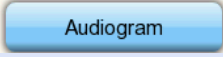

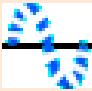
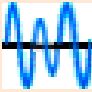



Symbols footer	Meaning
 	<p>Stimulus output on right ear (red).</p> <p>Stimulus output on left ear (blue).</p>
SUN Training	Special symbol during SUN Training test:
	<p>The training phase can be switched to test mode immediately. The symbol is in the hidden footer, which can be shown by pressing the on / off switch of the instrument (see Fig. 2). In test mode, all log-atomes are presented with increasing noise level in order to test speech understanding in noise.</p>

Symbols in the main screen	Meaning
	<p>Image-based, self-controlled pure-tone audiometry for children from 3 years on. There are two test types: MAGIC Audio and MAGIC Screen. The footer will be removed when using MAGIC (see 3.1). Advices of how to instruct children to perform the test as well as more details of the measuring procedure can be found in the How-To Manual / Chapter 2 'MAGIC'.</p>
	<p>MAGIC audiometry mode: Frequencies from 250 Hz to 8 kHz, initial stimulus level and stimulus type can be chosen.</p>
	<p>MAGIC screening mode: Frequencies from 250 Hz to 8 kHz, screening level and stimulus type can be chosen.</p>
	<p>Conventional pure-tone audiometry following ISO 60645-1: Class 4 (screening up to 70 dB HL) or class 3 (diagnostic up to 100 dB HL for air- and bone-conduction; insert sound probes, patient response switch, contralateral masking, stimulus selection). For more details about measuring procedures see How-To Manual / Chapter 3 'PTA'.</p>

Symbols in the main screen	Meaning
	<p>Screening test for assessing speech intelligibility in noise in school children and adults. Vocal-Consonant-Vocal logatomes are used. The test is available for different languages (I, D, E, F). The screening level can be chosen between 50 and 70 dB HL. Sound presentation is available via headphones, insert earphones or free-field loudspeakers.</p> <p>For more details about the measuring procedure see How-To Manual / Chapter 5 'SUN'.</p>
	<p>In training mode, all logatomes are presented without noise. This is intended for instructional purposes.</p>
	<p>In test mode, all logatomes are presented with increasing noise level in order to test speech understanding in noise.</p>
	<p>View results of stored measurements of the selected patient.</p>
<p>Symbols of result view</p>	<p>The test results shall always be interpreted by an expert. The following symbols are only meant as visual indicators and thus do not imply any diagnostic recommendation.</p>
	<p>Test result OK.</p>
	<p>Test result not OK.</p>
	<p>Test result needs to be seen in detail to decide if OK (e.g. aborted measurement). Result might be in-between OK and not OK.</p>
<p>Test names and layout</p>	<p>The following abbreviations are used for the different test results: MAGIC (Audiogram mode) PTA (Audiogram) SUN (Score result)</p> <p>Screening tests/modes are given with a 3-letter abbreviation and the stimulus level: MAG45 (MAGIC Screening at 45 dB HL)</p> <p>Additionally, the tested ear (right, left), the date and time of the measurement, and a visual indicator of the test result is given.</p>

Symbols in the main screen	Meaning
Additional symbols	To start a measurement, change settings...
	Start test with right ear.
	Start test with left ear.
 	Start test for both ears (binaural or serial processing right and left ear).
	Decrease value (e.g. frequency, level).
	Increase value (e.g. frequency, level).
 	Check box: multiple selections possible.
 	Radio button: single selection from the radio button group possible.
MAGIC Test	Special symbols used in MAGIC test:
	Restart instruction phase.
	Hide stimulus information in header (button toggles between hide and show).
	Show audiogram (intermediate result).

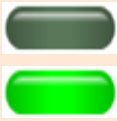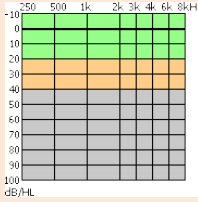Symbols in the main screen	Meaning
 e.g.	Different animals in the MAGIC test represent different frequencies.
 e.g.	Tone on (while button is pressed).
 e.g.	Animal with scarf: response symbol for tone “not heard”.
 e.g.	Animal without scarf: response symbol for tone “heard”.
	Repeat MAGIC audiogram test at selected frequencies.
	If a “muted stimulus” was “heard”, this might be an indicator for reduced attentiveness. The number of these “wrong” responses is shown in audiogram mode at the respective frequency beside the question mark symbol. The measurement at these frequencies should be repeated.
PTA Test	Special symbols during PTA test (Pure Tone Audiometry):
	The stimulus is presented as long as the loudspeaker button is pressed.
	Decrease / Increase level.
	Stimulus / Noise indicator: Lights highlighted as long as the stimulus (orange light) or noise (green light) is presented.

Symbols in the main screen	Meaning
	<p>Patient response indicator: if the patient response button is pressed the indicator is highlighted (green light).</p>
	<p>The threshold at the crosshairs can be set by clicking on the audiogram.</p>
	<p>Use continuous sine tone as stimulus.</p>
	<p>Use pulsed sine tone as stimulus.</p>
	<p>Use warble tone as stimulus.</p>
	<p>Use air conduction.</p>
	<p>Use bone conduction placed at the forehead.</p>
	<p>Use bone conduction placed at the mastoid.</p>

3 Start, reset, charging, and connecting sockets

3.1 On / off switch - special function for showing footer

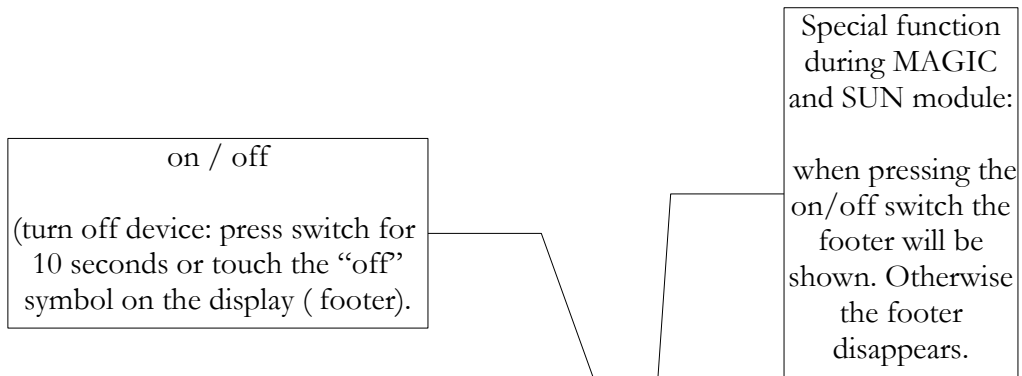


Fig. 2: On / off switch on the right-hand side of the device; special functionality during MAGIC and SUN module.

3.2 Hardware reset - device is stalled




Push the reset button below the rubber casing on the back side of the device with a pen. Afterwards the device can be turned on with the on switch.

Fig. 3: Black reset button on the back side of the device below the red rubber casing

3.3 Connecting to the sockets of the device

Blue socket

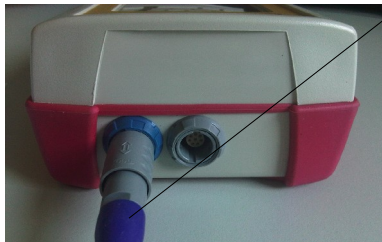

Headphone (for audiometry) plug with blue tension relief

Fig. 4: Blue and grey socket



Fig. 5: Free field loudspeakers JBL Control 2

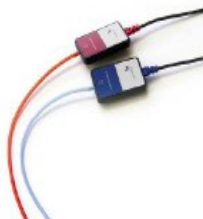


Fig. 6: GN otometrics insert earphones



Fig. 7: Headphone Interacoustics DD-45



Fig. 8: Headphone Holmco PD-81



Fig. 9: Headphone Sennheiser HDA 280

Grey socket



Patient response switch, bone conductor or charger plug with grey tension relief and labelprinter are to be connected to the grey socket.

Fig. 10: Power supply and charging cable with grey tension relief



Fig. 11: Patient response switch



Fig. 12: Patient response switch combined with bone conductor



Fig. 13: power plug



Fig. 14: Labelprinter

Serial interface cable

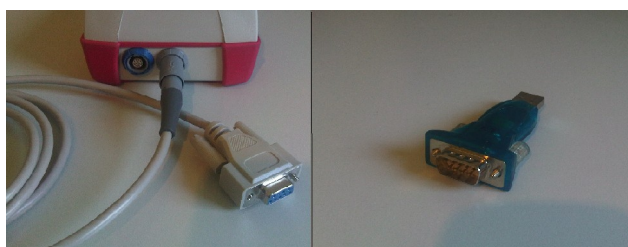


Fig. 15: left: serial interface cable RS232 with grey tension relief – right : USB converter

Connect the handheld device via a serial interface cable RS 232 to your PC (see Fig. 15 left) in order to update the device firmware or to exchange measurement data with the MIRA PC software (see Chapter 4).

If necessary use a 'serial to USB converter' in order to get connected to your PC (Fig. 15 right). Please refer to the manufacturer's information on the USB convertor's driver installation.

3.4 Charging the device and connecting to the label printer

Power supply

Connect the charging cable as seen in Fig. 10 to the device. For charging the device, connect the power plug to a power socket with appropriate output voltage and frequency (see data on charger). The charging process starts automatically and is finished within 2 hours.

Connect to label printer

When using the PRINTER module (see license management, section 5.5), you are able to print the test results directly from the device (View test menu). Therefore you need the Seiko Smart Label Printer 440 or 450 as well as a special connector cable to the device (Art. nr. 100 189). Please connect the cable to the device as pictured in Fig. 10.

The label printer must be connected to the device only with this special connector cable in order to maintain patient safety and integrity of the medical device.

4 MIRA - PC software and updates

4.1 Range of functions of the MIRA PC software

Irrespective of the installed licenses (i.e., modules) on the device, with the MIRA PC software you are able to update your device firmware. For updating your device firmware please connect the device via RS232 cable to your PC.

Devices which have the SW-COM module enabled (license) additionally can transfer data between the device and the PC. MIRA PC software simplifies data analysis, enables user configuration, and allows adding comments to patient and test data. Various report options simplify documentation (office printer). Please note that MIRA does not provide any additional diagnostic function.

4.2 How to get MIRA

The latest MIRA PC software and its corresponding manual are available via download on the PATH medical homepage.

Url: <http://www.pathme.de/support>

4.3 How to get updates

New device firmware and PC software updates will be posted each April and October on the PATH medical homepage.

Url: <http://www.pathme.de/support>

If any additional updates are available, the distributors will be informed. The distributors are supposed to inform the end customers.

5 First steps

5.1 User / patient selection

After turning on the device, you will be asked either to select a user **or** to select an existing patient or create a new patient data set. Dependent on your application situation it may be useful to activate or deactivate the user management (see Fig. 16 / 17). With the MIRA PC software (see Chapter 4) you are able to (de-)activate user management and to create different user profiles with or without password, which can be uploaded to the device.

If you need further information please use the context-sensitive help on the device (i.e., press the info icon in the footer).

Note:

It is assumed that one user will usually login and work with the device until turning off the device. Hence, changing a user is possible by turning off and on the device.



Fig. 16: Initial screen if no password was assigned or after login

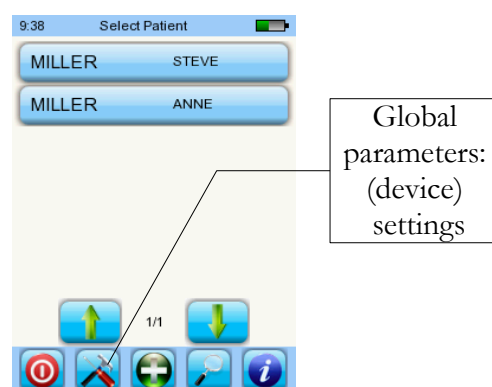






Fig. 17: Patient list if patient data had been generated

5.2 Device settings

The following settings can be changed on the device (see Fig. 17: global parameters):

- Date / Time (including date and time format: e.g. DD.MM.YYYY or MM/DD/YYYY)
- Language (selection out of several languages dependent on the installed language pack, i.g. E,D, I, ESP, F)
- Sound / Brightness
- Hardware tests
- Delete data (data will be removed from device – to restore data, do not delete before transferring data to the PC - see Chapter 4)
- System information

5.3 Hardware test and possible error messages

Device self test	Error message	Recommendation / Action
Battery / Core voltage		Please contact your distributor.
Codec		Please contact your distributor.
SDRAM		Please contact your distributor.
All tests		Selftest was successful. Status o.k.

5.4 System information and demo mode

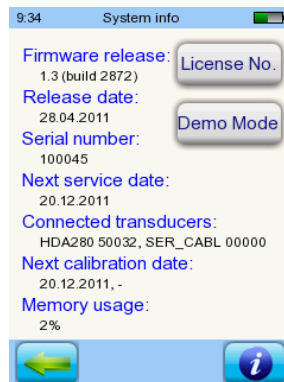


Fig. 18: System information with license management and demo mode

In the system information, general information about the device and firmware version is displayed. Information about connected transducers are also displayed (connected before menu is entered). When contacting your distributor (error message, module update...) this data should be at hand.

You can activate the demo mode 10 times. In demo mode, you are able to use all modules of your platform until the end of the day. If you are interested in upgrading your device with a specific module, please contact your distributor.

5.5 License management and upgrades

In order to update your license key (e.g., after buying a new module) you need to press the "License No." button on the System Information screen (see Fig. 18).

The already entered license key and all currently licensed modules are displayed. If you would like to add other modules to your device please contact your distributor. You can use the demo mode to evaluate the need for additional modules for your device (see section 5.4).

From the distributor you will receive a new license key to be installed on your device. Before installing a new license key, please make sure to have the former license key available in written (e.g. on delivery note) for potential reinstallation if needed.

Install a new license: Press the "Enter license key" button, enter your new license key and confirm the input by pressing the "ENTER" key.

5.6 Other errors and their possible reasons

Error description	Recommendation / Action
Black display.	The display is automatically deactivated after 2 minutes without user activity in order to increase use time without recharging. Please touch the display in order to leave the power saving mode.
No feedback, black display	After 5 minutes without user activity the device automatically powers down completely. Please start the device by pressing the on-switch.
No feedback, display stalled.	If the device does not respond to user action you might need to restart the device by pressing the reset switch (see Fig. 3). Please charge the battery if necessary. If the error is still present, please contact your distributor and describe which circumstances resulted in this behaviour.

5.7 Possible error messages during the measurement of...

Test	Error description	Recommendation / Action
MAGIC, PTA, SUN	Please make sure to have exactly one transducer type connected to the proper connector.	Please connect a headphone to the blue socket and if applicable a bone conductor to the grey socket.

5.8 Cleaning Senti

Before cleaning **Senti** the device must be turned off and removed from all connected devices.

Make sure that no liquids get to the interior of the device. Do not dunk the device into any liquid, e.g., water or cleaning agents.

Only use wiping disinfection cleaners (Ethanol: 70-80%, Propanol: 70-80%, or Aldehyde: 2-4%). Use a moistened cloth. Please adhere to the local regulations and laws.

5.9 Cleaning of headphone, accessory

Please follow the instructions of the manufacturer (see special data sheet of accessory).

6 Warranty, repair and service

6.1 Warranty

After the date of shipment of **Senti**, you are guaranteed the implied warranty for the statutory period. Warranty includes material and labor costs and has to be in accordance with the manufacturer specifications.

For the rechargeable battery pack, the touch screen and wearing parts, a six months period of warranty is provided. The warranty is only valid for devices purchased from an authorized distributor.

Warranty procedure: Inform your distributor about the defect. Send the device together with an error description to your distributor. Mailing expenses are not refundable and are to be paid by the customer. Please send the device in its original packaging to your distributor!

Warranty is not applicable in cases of breakage, malfunction due to manipulation or unintended usage, negligence, nonobservance of cleaning instructions, crashes or accidents, damages due to shipment. Warranty is also not applicable when the device is not used according to manufacturer's instructions.

6.2 Repair

In case **Senti** is defect or differs in any way from its original setup, an authorized distributor will repair, re-calibrate or exchange the device. Service features and repairs of the device and its electro-medical accessories must only be conducted by the manufacturer or its authorized service partners. The manufacturer reserves the right to decline any responsibility for the safety in operation, reliability, and capability of the device if any service features or repairs were conducted by a non-authorized body. If in doubt, please contact the manufacturer before making your service partner repair the device.

6.3 Service, routine maintenance

Calibration:

For all device types of the **Senti** device group, an **annual metrological inspection** following §11 Clause 2 of the medical device operator act must be conducted by a service partner which is authorized by the manufacturer.

Note: For the PTA module an annual inspection period is stipulated by the European standard EN 60645-1.

7 Accessories

Order Nr.	Article (alphabetical order)
100 119	Bone conductor Set (Bone conductor + Patient response switch + Firmware PTA 3)
100 214	Bone conductor with patient response switch BC-RE1 (available for PTA class 3 and PTA class 4 extended only)
100 083	Charger CH1
100 251	Free field cable
100 297	Free field loudspeaker Set (JBL Control 2) (loudspeaker + cable)
100 117	Headphone HP01 (Sennheiser HDA-280)
100 118	Headphone HP02 (Holmco PD81 circumaural)
100 282	Headphone HP03 (DD-45, similar to TD39)
100 273	Insert earphones (GN otometrics)
100 199	Label printer cable
100 189	Label printer with printer cable
100 169	Patient response switch
100 088	Serial interface cable SC1
100 162	Serial USB converter

8 Technical specifications, standards, manufacturer's data

8.1 Device classification and applied standards



Device class IIa (according to Directive 93/42/EEG Appendix IX)

Application part BF

Directive 93/42/EEG concerning medical products (1993, modified by 2007/47/EG)

German Medical Devices Act (Medizinproduktegesetz MPG) (2002)

EN ISO 9001 (2008)

EN ISO 13485 (2010)

EN ISO 14971 (2009)

EN ISO 10993-1 (2010)

EN 60601-1 (2007)

EN 60601-1-2 (2007)

EN 60601-1-4 (2001)

EN 60601-1-6 (2010)

EN 1041 (2008)

EN 980 (2008)

EN 60645-1 (2002) (PTA module)

All laws, directives and standards apply in their latest version.

8.2 Device, storage, transport

Device

Device dimensions: 209.3 x 98.0 x 34.8 mm
Weight (incl. Rechargeable battery pack and ear probe): 660 g
Real time clock
Rechargeable battery pack: duration of life > 2 years
Interfaces: RS232 up to 115 kbps;
Display: 240 x 320 pixel; graphic LCD 3.5"
Resistive touch screen
Up to five selectable languages per language pack
Power consumption: max. 2 W (400 mA)

Power supply / rechargeable battery

Power supply: auto backlight control; automatic shutoff; double voltage control
Maximum operating time with fully charged batteries: 6 hours
Rechargeable battery pack: 4.8 V NiMH
Input voltage: 100-240 V – AC 47-63 Hz, 0.16-0.29 A
Output voltage: 8-11 V; DC 12 W max.
Maximum charging cycles: 500 - 1000
Maximum charging time: 2 hours

Storage and transport

Please keep the device in the provided carrying case in order to protect the device and its accessories against external forces and environment impacts. Extreme storage and operating conditions may result in breakage of the touch screen display (extremely low temperature) or in impairment of the device's calibration.

Storage temperature: 0-40°C (32-104°F)
Operating temperature: 10-40°C (50-104°F)
Air humidity: 20-80% rel.
Air pressure: 900-1030 hPa

8.3 Modules

You will find further information with respect to the available modules in the How-To manual.

MAGIC

Frequencies: 0.25, 0.5, 1, 2, 3, 4, 6, 8 kHz
Stimulus levels: 5 to 70 dB HL (in steps of 5 dB)

PTA4

Screening audiometer class 4 according to EN 60645-1
Frequencies: 0.25, 0.5, 1, 2, 3, 4, 6 kHz
Stimulus levels: 0 to 70 dB HL (in steps of 5 dB)

PTA4 Extended

Screening audiometer class 4 according to EN 60645-1 with extended frequency/level range
Frequencies: 0.25, 0.5, 1, 2, 3, 4, 6, 8 kHz
Stimulus levels: -10 to max. 100 dB HL (in steps of 5 dB)

PTA3

Audiometer class 3 according to EN 60645-1
Frequencies: 0.25, 0.5, 1, 2, 3, 4, 6, 8 kHz
Stimulus levels: -10 to max. 100 dB HL (in steps of 5 dB)

SUN

Speech understanding in noise
Speech level: 50-70 dB HL (in steps of 5 dB HL)
Optional free field loudspeaker calibration via CCITT noise

8.4 Accessories

Accessories like e.g. headphones or insert earphones include separate manuals / data sheets which contain important information.

8.5 Electromagnetic compatibility report (EMC report)



Labor für die Prüfung der elektromagnetischen Verträglichkeit
Electromagnetic Compatibility Testing Laboratory

SCHWILLE - ELEKTRONIK
Produktions- und Vertriebs GmbH
Benzstrasse 1 A
85551 Kirchheim/ Germany

EMV Prüfbericht EMC Testreport

Gegenstand <i>Equipment (EUT)</i>	Audiologisches Handgerät
Hersteller <i>Manufacturer</i>	PATH medical GmbH 82110 Germering
Typ <i>Type</i>	Senti/ Sentiero
Auftraggeber <i>Customer</i>	PATH medical GmbH 82110 Germering
Anforderung <i>Requirement</i>	DIN EN 60601-1-2; VDE 0750-1-2:2007-12 Medizinische elektrische Geräte Teil 1-2: Allgemeine Festlegungen für die Sicherheit einschließlich der wesentlichen Leistungsmerkmale - Ergänzungsnorm: Elektromagnetische Verträglichkeit - Anforderungen und Prüfungen (IEC 60601-1-2:2007, modifiz EN 60601-1-2:2007 DIN EN 60645-1 Norm , 2002-09 Akustik - Audiometer - Teil 1: Reinton-Audi (IEC 60645-1:2001); Deutsche Fassung EN 60645-1:2001
Ergebnis <i>Result</i>	Die Übereinstimmung mit den Anforderungen ist erfüllt. <i>The compliance with the requirements is fulfilled.</i>
Gesamt <i>Total</i>	50 Seiten <i>50 pages</i>

Dieser Prüfbericht darf nur vollständig und unverändert weiterverbreitet werden. Auszüge und Änderungen | Genehmigung des ausstellenden Laboratoriums. Prüfberichte ohne Unterschrift und Stempel haben kein
Die Prüfergebnisse beziehen sich ausschließlich auf den Prüfgegenstand. Die Messgrößen und di
Kalibrierungen sind rückführbar auf nationale DKD Einheiten.

*This test report may not be reproduced other than in full except with the permission of the issuing labo
reports without signature and seal are not valid. This test report applies to the tested object only. The m
and annual calibration is traceable to national DKD normals.*

Fig. 19: Copy of the electromagnetic compatibility (EMC) report, bilingual Ger-
man / English


The EMC report certifies the conformity with respect to the mentioned requirements. **Senti** can be used in an environment with electromagnetic radiation as specified in the detail report (see fig. 20). The user shall take care, that the device is used in an environment with minimum distances to potential radiators as mentioned in fig. 21 (table with Nennleistung = effective power and Abstand = distance, dependant on frequency of radiator /sender).

Störaussendungs-Messungen	Übereinstimmung	Elektromagnetische Umgebung - Leitlinie
HF Aussendung nach CISPR 11	Gruppe 1	Das MEG verwendet HF-Energie ausschließlich zu seiner internen Funktion. Daher ist seine HF-Aussendung sehr gering und es ist unwahrscheinlich, dass benachbarte elektronische Geräte gestört werden.
HF Aussendung nach CISPR 11	Klasse B	Das MEG ist für den Gebrauch in allen Einrichtungen einschließlich denen im Wohnbereich und solchen, geeignet, die unmittelbar an ein öffentliches Versorgungsnetz angeschlossen sind, das auch Gebäude versorgt, die zu Wohnzwecken benutzt werden.
Aussendung von Oberschwingungen nach IEC 61000-3-2	Klasse A.	
Aussendungen von Spannungsschwankungen/ Flicker nach IEC 61000-3-3	Stimmt überein.	

Störfestigkeits-Prüfungen	IEC 60601-Prüfpegel	Übereinstimmungs-Pegel	Elektromagnetische Umgebung - Leitlinien
Entladung statischer Elektrizität (ESD) nach IEC 61000-4-2	± 8 kV Kontaktentladung ± 8 kV Luftentladung	± 8 kV Kontaktentladung ± 8 kV Luftentladung	Fußböden sollten aus Holz oder Beton bestehen oder mit Keramikfliesen versehen sein. Wenn der Fußboden mit synthetischem Material versehen ist, muss die relative Luftfeuchte mindestens 30 % betragen.
Schnelle transiente elektrische Störgrößen/ Bursts nach IEC 61000-4-4	± 2 kV für Netzleitungen ± 1 kV für Eingangs- und Ausgangsleitungen	± 2 kV für Netzleitungen ± 1 kV für Eingangs- und Ausgangsleitungen	Die Qualität der Versorgungsspannung sollte der einer typischen Geschäfts- oder Krankenhausumgebung entsprechen.
Stoßspannungen/ Surges nach IEC 61000-4-5	± 1 kV Spannung Außenleiter - Außenleiter	± 1 kV Spannung Außenleiter - Außenleiter	Die Qualität der Versorgungsspannung sollte der einer typischen Geschäfts- oder Krankenhausumgebung entsprechen.
Spannungseinbrüche, Kurzzeitunterbrechungen und Schwankungen der Versorgungsspannung nach IEC 61000-4-11	< 5 % U_T (> 95 % Einbruch der U_T) für 1/2 Periode 40 % U_T (60 % Einbruch der U_T) für 5 Perioden 70 % U_T (30 % Einbruch der U_T) für 25 Perioden < 5 % U_T (> 95 % Einbruch der U_T) für 5 s	< 5 % U_T (> 95 % Einbruch der U_T) für 1/2 Periode 40 % U_T (60 % Einbruch der U_T) für 5 Perioden 70 % U_T (30 % Einbruch der U_T) für 25 Perioden < 5 % U_T (> 95 % Einbruch der U_T) für 5 s	Die Qualität der Versorgungsspannung sollte der einer typischen Geschäfts- oder Krankenhausumgebung entsprechen. Wenn der Anwender des MEG fortgesetzte Funktion auch beim Auftreten von Unterbrechung der Energieversorgung fordert, wird empfohlen das EUT aus einer unterbrechungsfreien Stromversorgung oder einer Batterie zu speisen.
Magnetfeld bei der Versorgungsfrequenz (50 Hz/ 60 Hz) nach IEC 61000-4-8	3 A/m	3 A/m	Magnetfelder bei der Netzfrequenz sollten den typischen Werten, wie sie in der Geschäfts- und Krankenhausumgebung vorzufinden sind, entsprechen.

Anmerkung: U_T ist die Netzwechselspannung vor der Anwendung der Prüfpegel.

Fig. 20: Copy of electromagnetic compatibility detail report

Störfestigkeit s-Prüfungen	IEC 60601- Prüfpegel	Übereinstimmungs- Pegel	Elektromagnetische Umgebung - Leitlinien
Geleitete HF- Störgrößen nach IEC 61000-4-6	3 V Effektivwert 150kHz bis 80MHz	3 V eff	Tragbare und mobile Funkgeräte sollten in keinem geringeren Abstand zum EUTEinschließlich der Leitungen verwendet werden als dem empfohlenen Schutzabstand, der nach der für die Sendefrequenz zutreffenden Gleichung berechnet wird. Empfohlener Schutzabstand: $d = 3,5/3 * \text{Wurzel}(P)$
Gestrahlte HF Störgröße nach IEC 61000-4-3	3 V/m 80MHz bis 2,5GHz	3 V/m	$d = 3,5/3 * \text{Wurzel}(P)$ von 80 MHz bis 800 MHz $d = 7/3 * \text{Wurzel}(P)$ von 800 MHz bis 2500 MHz
			mit P als der Nennleistung des Senders in Watt (W) gemäß Angaben des Senderherstellers und d als empfohlenem Schutzabstand in Metern (m). Die Feldstärke stationärer Funksender sollte bei allen Frequenzen gemäß einer Untersuchung vor Ort ^a geringer als der Übereinstimmungspegel sein. ^b In der Umgebung von Geräten, die das Bildzeichen tragen, sind Störungen möglich.
			
Anmerkung 1:	Bei 80 MHz und 800 MHz gilt der höhere Frequenzbereich.		
Anmerkung 2:	Diese Leitlinien mögen nicht in allen Fällen anwendbar sein. Die Ausbreitung elektromagnetischer Größen wird durch Absorptionen und Reflexionen der Gebäude, Gegenstände und Menschen beeinflusst.		

Nennleistung des Senders W	Schutzabstand, abhängig von der Sendefrequenz		
	150 kHz bis 80 MHz	80 MHz bis 800 MHz	800 MHz bis 2,5 GHz
	$d = 3,5/3 * \text{Wurzel}(P)$	$d = 3,5/3 * \text{Wurzel}(P)$	$d = 7/3 * \text{Wurzel}(P)$
0,01	0,1	0,1	0,2
0,1	0,4	0,4	0,7
1	1,2	1,2	2,3
10	3,7	3,7	7,4

Fig. 21: Copy of electromagnetic compatibility detail report, distance to radiator/sender dependent on frequency

This is a blank page due to production process. Take your notes here:

Made in Germany by PATH medical GmbH

PATH medical GmbH
Landsberger Straße 63
82110 Germering
Germany
Tel. +49 89 800 76 502
Fax +49 89 800 76 503
<http://www.pathme.de>

CE
0124

