



Technical specifications, manufacturer's data

Senti & Sentiero

Manufacturer

PATH medical GmbH

Landsberger Straße 65
82110 Germering
Germany

Email ✉ info@pathme.de
Telephone ☎ +49 89 800 765 02
Fax +49 89 800 765 03

Manual Information

Release date: 2016-01-15
Revision: 0802_MA_TechnicalSpecifications_09
Valid from: Firmware Rev. 2.0, Mira PC Software Rev. 1.6.2

All mentioned items, products, brands and trademarks are registered or owned by the mentioned companies.

All information, illustrations, and specifications provided within this manual are based on the latest product information available at the time of publication. PATH medical reserves the right to make changes at any time without notice.

The latest revision of the Technical Manual is available online at www.pathme.de/support/download.

Errors and omissions excepted.

Copyright Notice

No part of this manual may be reproduced, translated, stored, or transmitted, in any form or by any means, electronic, mechanical, photocopying, recording, or otherwise, without the prior written consent of PATH medical GmbH.

Copyright © 2015 PATH medical GmbH



0124

Table of Contents

1.	General device information.....	4
1.1	Device	4
1.2	Power Supply, Rechargeable Battery	4
1.3	Storage, Transport, and Operating Conditions	4
2.	General transducer information.....	5
3.	Test modules	5
3.1	Multiple-Choice Auditory Graphical Interactive Check (MAGIC)	5
3.2	Pure-Tone Audiometry	6
3.3	Speech Understanding in Noise (SUN)	8
3.4	Mainzer Audiometric Test for Children (MATCH)	9
3.5	Munich Auditory Screening Test for Processing Disorders (MAUS).....	9
3.6	Bochum Auditory Speech Discrimination Test (BASD)	9
3.7	Universal Speech Test (UST)	10
3.8	Transient Evoked Otoacoustic Emissions (TEOAE)	10
3.9	Distortion Product Otoacoustic Emissions (DPOAE)	11
3.9.1	DPOAE Quick	12
3.9.2	DPOAE Diagnostic.....	12
3.9.3	DPOAE Threshold	12
3.10	Auditory Brainstem Responses (ABR)	13
3.10.1	ABR	13
3.10.2	Quick ABR	14
3.11	Auditory Steady-State Responses (ASSR)	14
3.12	Middle Ear Tests	15
3.12.1	Tympanometry	16
3.12.2	Acoustic Reflex	16
3.12.3	Eustachian Tube Function Tests	16
4.	Accessories	17

1. General device information

1.1 Device

- Device dimensions: 209 x 98 x 52 mm (Handheld), 150 x 210 x 45 mm (Desktop)
- Weight (incl. rechargeable battery pack): ca. 500 g (Handheld), ca. 475 g (Desktop)
- Display: 240 x 320 pixel; graphic LCD 3.5" (Handheld), 5.0" (Desktop)
- Features: resistive touch screen, real time-clock, piezoelectric sound generator
- Interfaces: RS232 (Senti), USB
- Output voltage and nominal impedance (headphone socket): 5 V_{pp}, 32 Ω
- Power consumption: max. 2 W

1.2 Power Supply, Rechargeable Battery

- Features: auto backlight control, automatic shutoff, double voltage control
- Rechargeable battery pack: 4.8 V NiMH (Handheld), 3.7 V Li-Ion (Desktop)
- Battery voltage limits for correct operation: 4.5 to 6.5 V (Handheld), 3.5 to 4.2 V (Desktop)
- Maximum operating time with fully charged batteries: ca. 6 to 8 hours (dependent on usage)
- Maximum charging cycles: 500 to 1000 (life time > 2 years for normal usage)
- Maximum charging time: ca. 2 hours (Handheld), ca. 8 hours (Desktop)
- Input rating of power supply units:
 - o Sinpro MPU12: 100-240 V, AC, 47-63 Hz, 0.16-0.29 A
 - o Sinpro MPU16: 100-240 V, AC, 47-63 Hz, 0.18-0.33 A
 - o Friwo 7662M/12 (GPP6): 100-240 V, AC, 50-60 Hz, 0.15 A
- Output rating of power supply units: 9V, 1.2 A (Handheld); 9-12 V, 0.4 A (Desktop)

1.3 Storage, Transport, and Operating Conditions

Please keep the device and its accessories in the provided carrying case in order to protect the device and its accessories against external forces and environmental impacts. Extreme storage and operating conditions may result e.g. in breakage of the touch screen display (extremely low temperature) or in impairment of the device's and transducer's calibration.

Transport and storage conditions:

- Transport temperature: -20 to 60 °C (-4 to 140 °F)
- Storage temperature: 0 to 40 °C (32 to 104 °F)
- Relative air humidity: ≤ 90 % non-condensing
- Barometric pressure: 50 to 106 kPa

Operating conditions:

- Temperature: 10 to 40 °C (50 to 104 °F)
- Relative air humidity: 20 to 90 % non-condensing
- Barometric pressure: 60* to 106 kPa

* At locations where the normal air pressure is below 80 kPa (at altitudes of more than 2000 meters), it is recommended to recalibrate the transducers.

Warm-up time: before turning on the device, make sure that the device is warmed up to room temperature, i.e. the operating conditions must be fulfilled.

2. General transducer information



Please note that maximum levels given below for each test module may vary dependent on the individual transducer calibration.

The contact pressure for the different transducers is described in *Table 1*:

Headphone	Pressure [N]
Sennheiser HDA-280	5.0 ± 0.5
Sennheiser HDA-200	10.0 ± 1.0
Sennheiser HDA-300	8.8 ± 0.5
Interacoustics DD-45	4.5 ± 0.5
Holmco PD-81	14.5 ± 1.0 *
GN Otometrics ME-70	4.5 ± 0.5
RadioEar B-71	5.4 ± 0.5
* No contact pressure tolerance provided in manufacturer's data sheet. Contact pressure tolerance given as for HDA-200.	

Table 1: Contact pressure for different transducers

For information about RETSPL values please refer to the default levels provided with the PATH Service Tool. For information about sound insulation properties of headphones please refer to the respective transducer data sheet.

3. Test modules

For further information about the test modules please refer to the "How-To Manual".

3.1 Multiple-Choice Auditory Graphical Interactive Check (MAGIC)

License: MAGIC

- Protocols: Screening, Audio
- Frequencies: 0.25 (cow), 0.5 (bear), 1 (elephant), 2 (cat), 3 (sheep), 4 (mouse), 6 (bird), 8 kHz (dolphin); optional: 5 kHz instead of 6 kHz
- Stimulus levels
 - o Screening: 20, 25, 30, 35, 40, 45, 60 dB HL
 - o Audio: -10 to max. 80 dB HL or transducer limits (see *Table 2*); step size: 5 dB
- Stimulus types: sine, warble tone (sine modulation, modulation depth = 10 %, $f_{\text{mod}} = 4$ Hz); single tone or two-tone presentation

Available transducers: Sennheiser HDA-280, Sennheiser HDA-200, Sennheiser HDA-300, Interacoustics DD-45, Holmco PD-81, GN ME-70, GN otoInsert, Etymotic Research ER-3A, RadioEar B-71

<i>f</i> [Hz]	250	500	1000	2000	3000	4000	5000	6000	8000
HDA-280	80	80	80	80	80	80	80	80	80
HDA-200	80	80	80	80	80	80	80	80	80
HDA-300	80	80	80	80	80	80	80	80	80
DD-45	80	80	80	80	80	80	80	80	80
PD-81	80	80	80	80	80	80	80	80	80
ME-70	80	80	80	80	80	80	80	80	80
otoInsert	80	80	80	80	80	80	80	80	70
ER-3A	80	80	80	80	80	80	80	80	75
B-71	35	50	60	60	60	50	45	40	35

Table 2: Maximum MAGIC sine stimulus levels for each transducer type

3.2 Pure-Tone Audiometry

Licenses: Audio Class 4: Screening audiometer (DIN EN 60645-1 class 4), Audio Class 4A: Screening audiometer (DIN EN 60645-1 class 4) with extended frequency/level range, Audio Class 3: Diagnostic audiometer (DIN EN 60645-1 class 3), Audio HF: frequency extension to any Audio license

- Protocols: Screening, Diagnostic; *Audio Class 3: + Automatic, Expert*
- Frequencies: 0.125, 0.25, 0.5, 0.75, 1, 1.5, 2, 3, 4, 5, 6 kHz; *Audio Class 4A/3: + 8 kHz; Audio HF: + 8, 9, 10, 11.2, 12.5, 14, 16 kHz*
- Stimulus levels:
 - o *Audio Class 4: 0 to max. 70 dB HL or transducer limits (see Table 3); step size: 5 dB*
 - o *Audio Class 4A/3: -10 to max. 110 dB HL or transducer limits (see Table 4); step size: 5 dB*
 - o *Audio HF: -10 (Audio Class 4A/3) or 0 (Audio Class 4) to max. 90 dB HL or transducer limits (see Table 4); step size: 5 dB*
- Stimulus types: sine, pulsed sine (repetition rate = 2.25 Hz), warble tone (sine modulation, modulation depth = 10 %, $f_{\text{mod}} = 4$ Hz)
- Minimum stimulus duration (optional): 1.2 s
- Masking noise (optional for *Audio Class 3: Expert*): third octave noise (see Table 5 and Table 6)
- Automatic testing (optional for *Audio Class 3: Auto*): Békésy (rate of level change = 5 dB/s), Hughson-Westlake (response window: start of stimulus+200 ms until end of stimulus), stimulus level limited to max. 80 dB HL, workflow according to ISO 8253-1, 6.2 and 6.3
- Uncomfortable level testing (optional for *Audio Class 3*)

Available transducers: Sennheiser HDA-280, Sennheiser HDA-200, Sennheiser HDA-300, Interacoustics DD-45, Holmco PD-81, GN ME-70, GN otoInsert, Etymotic Research ER-3A, JBL Control 2P (warble tone instead of sine); *Audio Class 3: + RadioEar B-71*

Available Audio HF transducers: Sennheiser HDA-200, Sennheiser HDA-300

f [Hz]	125	250	500	750	1000	1500	2000	3000	4000	5000	6000
HDA-280	60	70	70	70	70	70	70	70	70	70	70
HDA-200	60	70	70	70	70	70	70	70	70	70	70
HDA-300	60	70	70	70	70	70	70	70	70	70	70
DD-45	60	70	70	70	70	70	70	70	70	70	70
PD-81	60	70	70	70	70	70	70	70	70	70	70
ME-70	60	70	70	70	70	70	70	70	70	70	70
otoInsert	60	70	70	70	70	70	70	70	70	70	70
ER-3A	60	70	70	70	70	70	70	70	70	70	70
Control2P	---	---	70	---	70	---	70	70	70	---	---

Table 3: Maximum Audio Class 4 sine stimulus levels for each transducer type

f [Hz]	125	250	500	750	1000	1500	2000	3000	4000	5000	6000	8000
HDA-280	60	80	110	110	110	110	110	110	110	100	95	90
HDA-200	60	80	100	105	105	100	100	105	100	95	95	90
HDA-300	60	80	110	110	110	110	110	110	105	100	95	95
DD-45	60	80	100	110	110	105	105	110	105	100	95	90
PD-81	60	80	110	110	110	110	110	110	105	100	100	100
ME-70	60	80	100	105	105	105	105	105	105	100	100	90
otoInsert	60	80	100	105	110	105	105	105	105	90	80	70
ER-3A	60	80	105	110	110	110	110	110	105	95	85	75
Control2P	---	---	80	---	80	---	80	80	70	---	---	---
B-71 (Class 3)	---	35	50	55	60	60	60	60	50	45	40	35

f [Hz]	9000	10000	11200	12500	14000	16000
HDA-200	90	85	80	70	70	55
HDA-300	80	90	80	70	70	60

Table 4: Maximum Audio Class 3 and Audio HF sine stimulus levels for each transducer type

f_{center} [Hz]	125	250	500	750	1000	1500	2000	3000	4000	5000	6000	8000
f_L [Hz]	109	219	434	653	871	1307	1742	2613	3484	4353	5224	6965
f_U [Hz]	144	288	575	862	1149	1724	2299	3448	4597	5744	6893	9191

f_{center} [Hz]	9000	10000	11200	12500	14000	16000
f_L [Hz]	7838	8710	9759	10899	12221	14010
f_U [Hz]	10341	11491	12874	14376	16115	18461

Table 5: Audio Class 3 and Audio HF masking noise lower/upper band limits

<i>f</i> [Hz]	125	250	500	750	1000	1500	2000	3000	4000	5000	6000	8000
HDA-280	50	70	80	80	80	80	80	80	80	70	70	70
HDA-200	50	70	80	80	80	80	80	80	80	70	70	70
HDA-300	50	70	80	80	80	80	80	80	80	70	70	70
DD-45	50	70	80	80	80	80	80	80	80	70	70	70
PD-81	50	70	80	80	80	80	80	80	80	70	70	70
ME-70	50	70	80	80	80	80	80	80	80	70	70	70
otoInsert	50	70	80	80	80	80	80	80	80	70	70	70
ER-3A	50	70	80	80	80	80	80	80	80	70	70	70

<i>f</i> [Hz]	9000	10000	11200	12500	14000	16000
HDA-200	50	50	50	50	50	40
HDA-300	50	50	50	50	50	50

Table 6: Maximum Audio Class 3 and Audio HF masking noise levels for each transducer type

3.3 Speech Understanding in Noise (SUN)

License: SUN + language speech license

- Protocols:
 - o Fixed/predefined: constant speech level, adaptive noise level (SNR groups)
 - o Adaptive: adaptive speech level, constant noise level
- Speech level: 40 to max. 80 dB HL or transducer limits (see Table 7); step size: 5 dB
- Languages (pronunciations of logatoms): Italian, German, English; for validation: French, Spanish, Russian – additional languages upon request
- Character set: Latin, Greek, Farsi, Hindi, Cyrillic, Czech (not all character set / language combinations available)

Available transducers: Sennheiser HDA-280, Sennheiser HDA-200, Sennheiser HDA-300, Interacoustics DD-45, Holmco PD-81, GN ME-70, GN otoInsert, Etymotic Research ER-3A, JBL Control 2P (free field loudspeaker calibration via CCITT noise)

Calibration type	IEC	ANSI
HDA-280	80	80
HDA-200	80	75
HDA-300	80	80
DD-45	80	80
PD-81	80	80
ME-70	80	80
otoInsert	80	70
ER-3A	80	75
Control 2P	80	70

Table 7: Maximum SUN speech levels for each transducer type

3.4 Mainzer Audiometric Test for Children (MATCH)

License: MATCH + language speech license

- Protocols:
 - o Fixed: constant speech level; optional with ipsilateral noise
 - o Adaptive: adaptive speech level; optional with ipsilateral noise
- Speech level: -10 to max. 100 dB or transducer limits (see *Table 8*); step size: 1 dB
- Languages: German; for validation: English, French, Spanish, Russian, Turkish, Italian, Slovenian – additional languages upon request
- Options: ipsilateral masking with fixed level (65 dB), manual item selection

Available transducers: Sennheiser HDA-280, Sennheiser HDA-200, Sennheiser HDA-300, Interacoustics DD-45, Holmco PD-81, GN ME-70, GN otolInsert, Etymotic Research ER-3A, JBL Control 2P (free field loudspeaker calibration via CCITT noise)

Calibration type	IEC	ANSI
HDA-280	100	85
HDA-200	90	80
HDA-300	100	85
DD-45	95	80
PD-81	100	90
ME-70	95	80
otolInsert	85	75
ER-3A	85	75
Control 2P	85	70

Table 8: Maximum MATCH speech levels (without ipsilateral noise) for each transducer type

3.5 Munich Auditory Screening Test for Processing Disorders (MAUS)

License: MAUS + speech license

Copyright: Westra Elektroakustik GmbH 2003/2004. Details in separate MAUS manual. Authors: A. Nikisch, C. Heuckmann, T. Burger

Available transducers: Sennheiser HDA-280, Sennheiser HDA-200, Sennheiser HDA-300, Interacoustics DD-45, Holmco PD-81, GN ME-70, GN otolInsert, Etymotic Research ER-3A

3.6 Bochum Auditory Speech Discrimination Test (BASD)

License: BASD + speech license

- Differentiation modes: Consonants (Ba/Ga, Ga/Ka), frequency, duration, sound intensity, amplitude modulation (modulation depth = 100 %, reference modulation frequency = 20 Hz)
- Initial differences: sound intensity (5 to 20 dB; step size: of 5 dB), duration (100 to 300 ms; step size: 50 ms, frequency (0.05, 0.1, 0.2, 0.4, 0.8, 1.6 Oct), amplitude modulation (80 Hz)
- Presentation modes: Monaural, binaural, dichotic, interaural
- Stimulus level: 30 to max. 100 dB HL; step size: 5 dB

- Frequency (not for consonant differentiation tests): 0.5, 1 kHz

Available transducers: Sennheiser HDA-280, Sennheiser HDA-200, Sennheiser HDA-300, Interacoustics DD-45, Holmco PD-81, GN ME-70, GN otoInsert, Etymotic Research ER-3A

3.7 Universal Speech Test (UST)

License: UST + sub-module speech license

- Speech level: 0 to max. 100 dB or transducer limits (see *Table 9*); step size: 5 dB
- Word lists: UT01: Dr. Tato (Spanish); UT02: Freiburger (German); UT06: Govorni (Slovenian); UT09: Hungarian speech test; UT08: Spondees child, UT10: Spondees adult, UT11: NU-6, UT12: CID W-22, UT13: PBK (English) – additional word lists upon request
- Contralateral or ipsilateral noise (optional): fixed (0 to max. 100 dB), offset (-30 to +30 dB)
- Pre-test phases (optional): voice hearing, word understanding

Available transducers: Sennheiser HDA-280, Sennheiser HDA-200, Sennheiser HDA-300, Interacoustics DD-45, Holmco PD-81, GN ME-70, GN otoInsert, Etymotic Research ER-3A, JBL Control 2P (free field loudspeaker calibration via CCITT noise)

Calibration type	IEC	ANSI
HDA-280	100	85
HDA-200	90	80
HDA-300	100	85
DD-45	95	80
PD-81	100	90
ME-70	95	80
otoInsert	85	75
ER-3A	85	75
Control 2P	85	70

Table 9: Maximum UST speech levels (without ipsilateral noise) for each transducer type

3.8 Live Speech

License: Live Speech

- Level setting: 0 to 80 dB HL; step size: 5 dB
- Options: microphone sensitivity adjustment (according to VU meter), heard/not heard entry

Available transducers: Sennheiser HDA-280, Sennheiser HDA-200, Sennheiser HDA-300, Interacoustics DD-45, Holmco PD-81, GN ME-70, GN otoInsert, Etymotic Research ER-3A

3.9 Transient Evoked Otoacoustic Emissions (TEOAE)

License: TEOAE

- Modules: Quick, Diagnostic

- Noise detection: root mean square (RMS) of non-stimulus intervals
- Residual noise calculation: weighted averaging, summed weighting factors
- Artifact rejection: weighted averaging
- Response detection:
 - o TEOAE Quick: 8 values with changing sign fulfilling a 3 sigma criterion (representing 99.7 % statistical significance)
 - o TEOAE Diagnostic: user-defined stop criterion (SNR: 6 or 9 dB) in 3, 4, or 5 out of 5 frequency bands (1, 1.5, 2, 3, 4 kHz)
- Leak check: analysis of feedback signal (440 Hz probe tone)
- Probe check: limit of maximum sound pressure (“stimulus”), comparison across speakers (“symmetry”), leak check (“probe fit”)
- Calibration: in-the-ear calibration with ear canal volume adjustment
- Sample rate: 48 kHz (stimulus), 16 kHz (response)
- Window of analysis: 5 to 13 ms post-stimulus
- Simultaneous measurement on left/right ear possible when connecting two ear probes

- Stimulus level:
 - o TEOAE Quick: 85 dB peSPL
 - o TEOAE Diagnostic: 60 to 85 dB peSPL; step size: 5 dB
- Stimulus types: short-term stimulus without steady component (0.7-6 kHz); TEOAE Diagnostic: + 100 μ s rectangular click
- Stimulation protocol: nonlinear
- Test time (TEOAE Diagnostic): 15 s, 30 s, 60 s, Automatic

Available transducers: PATH EP-TE, PATH EP-DP, PATH EP-VIP*, PATH EP-TY*

* Available for Sentiero Desktop, without status light functionality for Sentiero⁺, Sentiero Advanced

3.10 Distortion Product Otoacoustic Emissions (DPOAE)

License: DPOAE Quick/Diagnostic, DPOAE Threshold; feature upgrades: FMDPOAE, DPHIRES

- Modules: Quick, Diagnostic, Threshold (for details see separate module sections)
- Noise detection: narrow band noise around $2f_2-f_1$
- Residual noise calculation: weighted averaging, summed weighting factors
- Artifact rejection: weighted averaging
- Response detection: F-test, F-value at a single point (F_{sp}) dependent on settings
- Leak check: analysis of feedback signal (440 Hz probe tone)
- Probe check: limit of maximum sound pressure (“stimulus”), comparison across speakers (“symmetry”), leak check (“probe fit”)
- Calibration: in-the-ear calibration with ear-canal volume adjustment
- Frequency ratio f_2/f_1 : 1.22
- Minimum DPOAE level criterion: $L_1 - 70$ dB
- Sample rate: 48 kHz (stimulus, response)
- Measurement interval: 4096 samples
- Stimulus modes (with *FMDPOAE* license):
 - o Frequency-modulated DPOAE ($f_m = 1.4-1.6$ Hz, modulation depth = 100 Hz at 4 kHz)

- Multi-channel DPOAE (simultaneous measurement of DPOAEs at up to two f_2 frequencies at a time)
- Simultaneous measurement on left/right ear possible when connecting two ear probes

Available transducers: PATH EP-DP, PATH EP-VIP*, PATH EP-TY*

* Available for Sentiero Desktop, without status light functionality for Sentiero⁺, Sentiero Advanced

3.10.1 DPOAE Quick

License: DPOAE Quick/Diagnostic; feature upgrade: FMDPOAE

- Frequencies f_2 : 1, 1.5, 2, 3, 4, 5, 6, 8 kHz
- Stimulus level L_2 : 30 to 65 dB SPL; step size: 5 dB
- L_2/L_1 relation: automatic (scissor paradigm: $L_1 = 0.4 L_2 + 39$ dB SPL, Kummer et al. 1998)
- SNR stop criterion: 6, 9, 12 dB
- Overall stop criterion: x out of y (with y = number of selected frequencies, $x = y/y-1/y-2$ & $x > y/2$) with “as fast as possible” option, i.e. stop as soon as overall criterion is fulfilled or cannot be fulfilled anymore
- Maximum number of recalibrations until stop: 0, 1, 3, 10
- Manual retest

3.10.2 DPOAE Diagnostic

License: DPOAE Quick/Diagnostic; feature upgrades: FMDPOAE, DPHIRES

- Frequencies f_2 (standard): 1, 1.5, 2, 3, 4, 5, 6, 8 kHz
- Frequencies f_2 (with DPHIRES license):
 - Linear: 0.8 to 10 kHz (step size: 0.5 kHz from 1 to 10 kHz), steps: 10 to 1000 Hz (step size: 10 Hz, minimum step size depends on start and stop frequency)
 - Logarithmic: 0.8 to 10 kHz (step size: 0.5 kHz from 1 to 10 kHz), steps: 1 to 30 points per octave (step size: 1 point per octave)
- Stimulus levels L_2 : 30 to 65 dB SPL; step size: 5 dB (single and multiple selections possible)
- L_2/L_1 relation: automatic (scissor paradigm), $L_1=L_2$, $L_1=L_2+5$ dB, $L_1=L_2+10$ dB (max. L_1 limited to 70 dB SPL)
- SNR stop criterion: 6, 9, 12 dB
- Minimum DPOAE level criterion (optional): -20 to 0 dB; step size: 5 dB
- Measurement time: adaptive timeout, manual minimum/maximum timeout (2 to 120 s)
- Options: automatic retest; static pressure offset (Sentiero Desktop only)

3.10.3 DPOAE Threshold

License: DPOAE Threshold; feature upgrade: FMDPOAE

- Frequencies f_2 : 1, 1.5, 2, 3, 4, 5, 6, 8 kHz
- Stimulus level L_2 : 20 to 65 dB SPL (automated threshold detection algorithm)
- Minimum stimulus level L_2 : 20, 25, 30 dB SPL
- L_2/L_1 relation: automatic (scissor paradigm)

3.11 Auditory Brainstem Responses (ABR)

License: ABR; feature upgrades: ABR-BIN, ABR-FS

- Modules: Standard, Quick ABR (for details see separate module sections)
- Artifact rejection: weighted averaging, notch filter (50/60 Hz, self-tuning)
- Residual noise calculation: collecting noise energy from each frame, calculating residual noise level (absolute RMS value in nV)
- Response detection: auto peak-marker setting via template matching
- Normative latencies for different age groups and transducer types
- Display and storage of waveform, impedance, residual noise, averages; standard mode: + peak marker (editable)
- Electrode impedance check:
 - o Continuous monitoring of electrode impedance
 - o Auto start after impedance OK (optional): $R \leq 4 \text{ k}\Omega$, $\Delta R \leq 2 \text{ k}\Omega$
 - o Allow manual start: $R \leq 6 \text{ k}\Omega$, $\Delta R \leq 3 \text{ k}\Omega$; allow skip: $R \leq 12 \text{ k}\Omega$, $\Delta R \leq 6 \text{ k}\Omega$; stop during test: $R > 7 \text{ k}\Omega$, $\Delta R > 4 \text{ k}\Omega$; stop during test (if skipped): $R > 13 \text{ k}\Omega$, $\Delta R > 7 \text{ k}\Omega$
- Sample rate: 48 kHz (stimulus), 16 kHz (response)
- Simultaneous measurement on left/right ear (with ABR-BIN license)

3.11.1 ABR

License: ABR; feature upgrades: ABR-BIN, ABR-FS

- Stimulus types: Click (0.7 to 6 kHz), Chirp (broadband, 1 to 8 kHz); with ABR-FS license: + Low-Chirp (100 to 850 Hz), Mid-Chirp (850 Hz to 3 kHz), High-Chirp (3 to 10 kHz), Tone Burst (500 Hz, 750 Hz, 1 kHz, 1.5 kHz, 2 kHz, 3 kHz, 4 kHz; waveform: 2 up, 1 plateau, 2 down)
- Stimulus polarity: condensation, rarefaction, alternating
- Stimulus rate: 10, 20, 30, 40, 70, 80, 90 Hz
- Stimulus levels: 0 to max. 90 dB nHL or transducer limits (see Table 10), no stimulus; step size: 5 dB; single or multiple selection of up to five levels per test sequence
- Masking offset levels (white noise): -50 to +50 dB
- Averages: 1000 up to 20000; step size: 1000
- Noise stop criterion (optional): 10, 15, 20 to 80 nV; step size: 10 nV
- Automated wave 5 detection (optional) with optional minimum wave 5 criterion: 20, 30, 40, 50, 70, 100, 150, 200 nV_{pp}
- Plot range (fixed): 0 to inter-stimulus interval + 1.5 ms (minimum 10.5 ms, maximum: 15 ms)
- Additional parameters: Jitter, Auto Proceed, Auto Stop

Available transducers: Sennheiser HDA-280, Sennheiser HDA-200, Sennheiser HDA-300, Interacoustics DD-45, Holmco PD-81, GN ME-70, GN otoInsert, Etymotic Research ER-3A, PATH EP-TE, PATH EP-DP, PATH EP-VIP*, PATH EP-TY*, RadioEar B-71

* Available without status light functionality for Sentiero Advanced

Stim. Type	Click	Chirp	Low-Chirp	Mid-Chirp	High-Chirp
HDA-280	85	90	90	90	85
HDA-200	80	85	85	85	80
HDA-300	85	90	90	90	85

DD-45	85	90	85	90	85
PD-81	85	90	90	90	80
ME-70	80	85	85	85	85
otoInsert	80	85	85	90	75
ER-3A	85	90	85	90	80
EP-DP ^(R1) *	80	85	70	90	80
^(R2)	85	90	80	90	75
EP-VIP, -TY ^(R2) *	90	90	85	90	80
B-71	65	65	65	70	55

* Example adult ear. Dependent on ear canal volume, the actual level may be lower (big ear canal volume) or higher (small ear canal volume). R1: SN40xxx, 50xxx, 80xxx; R2 (updated loudspeaker setup): SN41xxx/42xxx, 51xxx/52xxx, 81xxx/82xxx.

Stim. type	TB 0.5	TB 0.75	TB 1k	TB 1.5k	TB 2k	TB 3k	TB 4k
HDA-280	90	90	90	90	90	90	90
HDA-200	80	80	85	85	90	90	90
HDA-300	90	90	90	90	90	90	90
DD-45	85	85	90	90	90	90	90
PD-81	90	90	90	90	90	90	90
ME-70	80	85	90	90	90	90	90
otoInsert	85	85	90	90	90	90	90
ER-3A	85	90	90	90	90	90	90
EP-DP ^(R1) *	60	65	65	75	75	85	85
^(R2)	70	75	75	80	80	85	80
EP-VIP, -TY ^(R2) *	75	80	80	90	85	90	90
B-71	65	70	70	65	65	70	70

* Example adult ear. Dependent on ear canal volume, the actual level may be lower (big ear canal volume) or higher (small ear canal volume). R1: SN40xxx, 50xxx, 80xxx; R2 (updated loudspeaker setup): SN41xxx/42xxx, 51xxx/52xxx, 81xxx/82xxx.

Table 10: Maximum ABR stimulus levels for each transducer type

3.11.2 Quick ABR

License: ABR; feature upgrade: ABR-BIN

- Stimulus type: Chirp (broadband, 1 to 8 kHz)
- Stimulus polarity: alternating
- Stimulus rate: 85 Hz

Stimulus level: 35 to 55 dB eHL (step size: 5 dB), ask before test (enables additional stimulus levels 25 and 30 dB eHL)

Please note: dB eHL = dB nHL + 10 dB

3.12 Auditory Steady-State Responses (ASSR)

License: ASSR

- Protocols:

- Fixed level procedure
- Adaptive level procedure (threshold estimation)
- Stimulus rate: 40.5 ± 2 Hz (40 Hz ASSR) and 86 ± 2 Hz (80 Hz ASSR)
- Stimulus level:
 - Fixed: 10 to max. 100 dB HL or transducer limits (see *Table 11*); single or multiple level selections possible; step size: 10 dB
 - Adaptive: 0 to max. 100 dB HL or transducer limits (see *Table 11*); step size: 10 dB
- Frequencies: 0.25, 0.5, 1, 1.5, 2, 3, 4, 6 kHz
- Number of averages: 45 to 900 s; step size: 15 s
- Noise stop criterion: 0 to 20 nV; step size: 1 nV
- Display and storage of waveform, impedance, artifact threshold, modulation frequency
- Electrode impedance check: see ABR

Available transducers: Sennheiser HDA-280, Sennheiser HDA-200, Sennheiser HDA-300, Interacoustics DD-45, Holmco PD-81, GN ME-70, GN otolInsert, Etymotic Research ER-3A, PATH EP-DP, PATH EP-VIP*, PATH EP-TY*

* Available without status light functionality for Sentiero Advanced

<i>f</i> [Hz]	250	500	1000	1500	2000	3000	4000	6000
HDA-280	90	100	100	100	100	100	90	80
HDA-200	80	90	90	90	90	90	80	80
HDA-300	90	100	100	90	90	100	90	80
DD-45	80	90	90	90	90	100	90	80
PD-81	90	100	100	100	100	100	90	90
ME-70	80	90	90	90	90	90	90	90
otolInsert	80	90	90	90	90	90	90	70
ER-3A	80	90	100	100	100	100	90	70
EP-DP ^(R1) *	80	70	70	80	80	90	90	80
^(R2)	80	80	80	90	90	90	90	70
EP-VIP, -TY ^(R2) *	90	90	90	90	100	100	100	80

* Example adult ear. Dependent on ear canal volume, the actual level may be lower (big ear canal volume) or higher (small ear canal volume). R1: SN40xxx, 50xxx, 80xxx; R2 (updated loudspeaker setup): SN41xxx/42xxx, 51xxx/52xxx, 81xxx/82xxx.

Table 11: Maximum ASSR stimulus levels for each transducer type (single frequency measurement only – simultaneous measurement at multiple frequencies may reduce the maximum stimulus level).

3.13 Middle Ear Tests

License: Tymp Class 1, Tymp Class 2; feature upgrade for Tymp Class 2: Tymp 1k

- Probe tone: 226 Hz ± 1 % at 85.3 dB SPL ± 3 dB
- Artefact detection

Available transducers: PATH EP-TY

3.13.1 Tympanometry

License: Tymp Class 1, Tymp Class 2; feature upgrade for Tymp Class 2: Tymp 1k

- Additional probe tones (class 1 only): 678 Hz \pm 1 % at 72 dB SPL \pm 3 dB, 800 Hz \pm 1 % at 70.6 dB SPL \pm 3 dB, 1000 Hz \pm 1 % at 69 dB SPL \pm 3 dB (1000 Hz can be added to *Tymp Class 2* license with *Tymp 1k* license)
- Multi-frequency measurement (*Tymp Class 1*: 226, 678, 800, 1000 Hz, *Tymp Class 2* + *Tymp 1k*: 226, 1000 Hz)
- Compliance Range: 0 to 5 ml
- Compliance Accuracy : \pm 5 % or 0.1 ml (whichever is greater)
- Pressure Range: +300 to -300 daPa (*Tymp Class 2*), +400 to -600 daPa (*Tymp Class 1*); minimum: -600 to -100 daPa, maximum: +200 to +400 daPa; step size: 50 daPa
- Pressure Accuracy: \pm 10 % or 10 daPa (whichever is greater)
- Pump speed: 50, 100, 150, 200 daPa/s \pm 10 daPa/s, as fast as possible
- Tympanometry + Acoustic Reflex sequence (see [3.13.2 Acoustic Reflex](#))
- *Tymp Class 1* options:
 - o Y (admittance) / B (susceptance) / G (conductance) components view
 - o Auto Stop function (finish recording if valid peak is detected)
 - o Multiple trace memory (up to three traces in one measurement)

3.13.2 Acoustic Reflex

License: Tymp Class 1, Tymp Class 2

- Modes: automatic reflex threshold; *Tymp Class 1*: + manual reflex test, reflex decay test
- Automatic reflex screening levels: 70 to 95 dB HL; step size: 5 dB
- Automatic reflex start after tympanometry options: never, always, if peak within norm (for 226 Hz probe tone only)
- Stimulus presentation modes: ipsilateral, contralateral (if 2nd transducer is connected)
- Pressure offset: taken over from tympanometry measurement, editable (see pressure range in section [3.13.1: Tympanometry](#))
- Acoustic reflex stimuli (using EP-TY probe): 500, 1000, 2000, 3000, 4000 Hz, up to 100 dB HL; Broadband noise (CCITT) up to 90 dB HL; low-pass noise (LPN: 871-1149 Hz), high-pass noise (HPN: 3484-4597 Hz) up to 90 dB HL
- Acoustic reflex stimulus duration: 2 s, 14 s for reflex decay
- Acoustic reflex stimulus signal time multiplexed with probe tone (106 ms on, 53 ms off)

Available transducers for contralateral reflex: Sennheiser HDA-280, Sennheiser HDA-200, Sennheiser HDA-300, Interacoustics DD-45, Holmco PD-81, GN ME-70, GN otoInsert, Etymotic Research ER-3A, Etymotic Research ER-3C, RadioEar IP-30, Audio Mouldings GBE, PATH EP-VIP

3.13.3 Eustachian Tube Function Tests

License: Tymp Class 1

- Modes:

- Non-perforated ear drum (Williams test)
- Perforated ear drum (Toynbee test)
- Patulous Eustachian tube (continuous sensitive impedance measurement)

4. Accessories

Accessories (e.g. headphones, insert earphones, electrode cables, electrodes, label printer) may include separate manuals and/or data sheets which may contain important information. Please refer to these documents for further information about the respective accessory.

Contact information from distributor/service partner:

Made in Germany



PATH medical GmbH
Landsberger Straße 65
82110 Germering
Germany

Tel.: +49 89 800 765 02 Fax: +49 89 800 765 03 Internet: www.pathme.de

CE

0124